



**NL Health
Services**

Quick Reference Guide

Using the VersaScope Exam Camera in a Webex Call

Make sure the VersaScope is charged and connected to the Telehealth computer before using it during the Telehealth call.

1. Turn on the VersaScope
Press and hold the power button on the top of the camera. You may hear one or two beeps when it connects to the computer.
2. Select the VersaScope in Webex (first time only)
 - In Webex, click the down arrow next to Stop Video
 - Select DriverCoding UVC Camera from the list (see image below)
3. During use
 - When the VersaScope is turned on, Webex should switch to it automatically
 - If it does not, click the down arrow next to Stop Video and select DriverCoding UVC Camera
4. After the exam
 - Turn off the VersaScope. Webex will switch back to the webcam automatically.

See the screenshot below for an example. Camera options may vary depending on your setup.

